# SG Link Set

OM-E0638E

## OPERATION MANUAL

Thank you for purchasing the NSK SG Link Set.

This product is a link device which enables a single Foot Control to operate the VarioSurg3 Piezo Surgical System and the Surgic Pro Surgical Micro Motor.

Please read this Operation Manual and the Operation Manuals of VarioSurg3 and Surgic Pro carefully before connecting for use. Proper Care & Maintenance is important for the long term durability of both products. Keep this Operation Manual within easy reach of users for future reference.

### – Contents –

1. Precautions for handling and operation	1
2. Package Contents	1
3. How to connect the parts	2
4. Operating method	5


## 1 Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
<b>NOTICE</b>	General product specification information highlighted to avoid product malfunction and performance reduction.

-  **CAUTION** • Do not use this product for purposes other than to link the NSK VarioSurg3 and NSK Surgic Pro.
- Prior to use, read this Operation Manual to properly familiarize yourself with the function of each part. We suggest that this Operation Manual is kept within easy reach of users for future reference.

## 2 Package Contents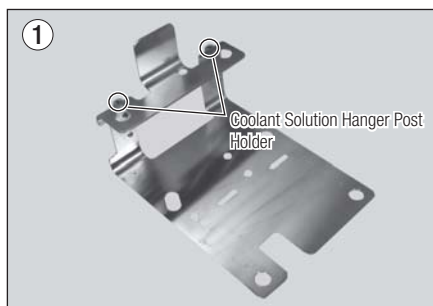

No.	Part name	Quantity
1	SG Link Stand	1
2	SG Link Cable	1

### 3 How to connect the parts

#### 3-1 Installation of the devices

- 1) Align the Rubber Pads on the base of the VarioSurg3 Control Unit with the Rubber Pad Insertion Points of the SG Link Stand (4 Points). (Fig. 1)

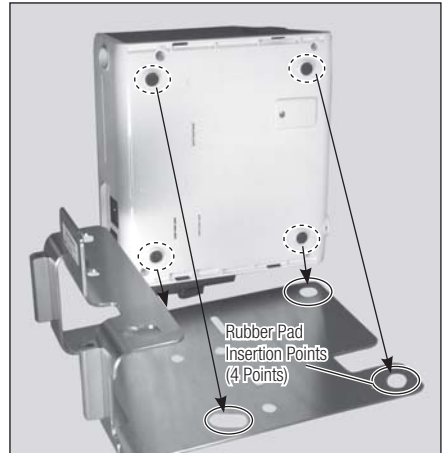
Fig. 1

- 2) Align the Rubber Pads on the base of the Control Unit of SurgicPro with the Rubber Pad Insertion Points of the SG Link Stand (2 Points). (Fig. 2)

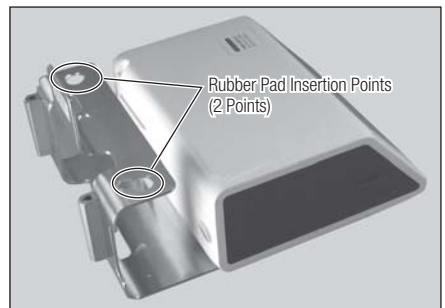
Fig. 2

**CAUTION**  • Make sure that Surgic Pro is placed ON TOP of VarioSurg3. If Surgic Pro is placed on the bottom, the torque load on the top face of Surgic Pro cannot be used.

#### 3-2 Connection of the parts

**CAUTION**  • Before connecting the AC power cord, turn OFF (○ side) the Power Switch.

- 1) Connect the AC power cords to the respective control units of VarioSurg3 and Surgic Pro.
- 2) Connect the Foot Control to the Control Unit of VarioSurg3. (Fig. 3)

Fig. 3

- 3) Open the cover of the SG Link Cable Socket of VarioSurg3 and insert the SG Link Connector plug all the way in. (Fig. 4)

Fig. 4

- 4) Insert the Foot Control Connector Plug of the SG Link Cable to the Foot Control Socket of the Surgic Pro and tighten the lock nut securely. (Fig. 5)

Fig. 5

**CAUTION**

- Always insert the SG Link Connector plug straight into socket/connector. If the plug is not correctly inserted the control unit may not operate or may cause damage to the SG Link Socket/Foot Control Cord Connector Socket.

- 5) Insert the Coolant Solution Hanger Posts belonging to VarioSurg3 and Surgic Pro to the holes of the Coolant Solution Hanger Post Holder, making sure that the guides of the Posts and the guide grooves are aligned. (Fig. 6)

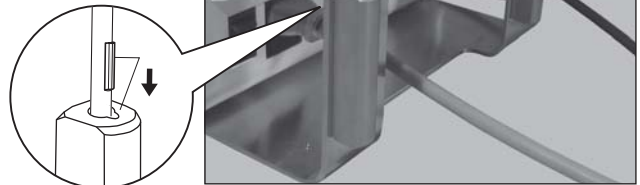
Fig. 6

(When the connection is finished)

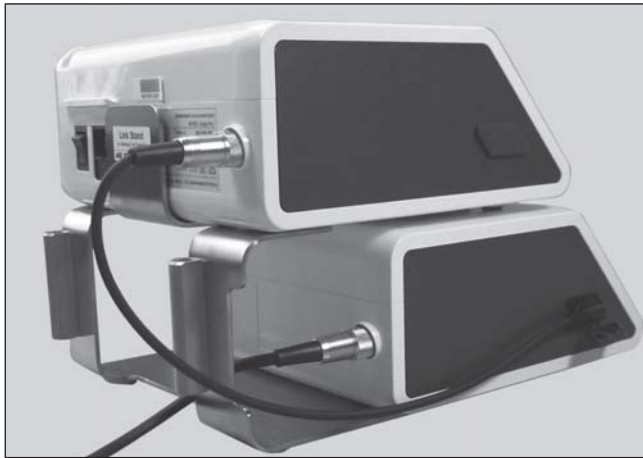
Fig. 7

**CAUTION**

- When SG Link Cable is not connected, close the cover of SG Link Cable Socket on the VarioSurg3.
- Do not connect SG Link Cable to the USB flash drive socket of the Surgic Pro+ as the USB terminal may be damaged.
- Avoid connecting and disconnecting SG Link Cable more than necessary as it may cause poor connection of the connector area resulting in malfunction.
- Make sure that the Coolant Solution Hanger Posts are inserted into the Coolant Solution Hanger Post Holder of the SG Link Stand.
- Connect the AC power cords of both Surgic Pro and VarioSurg3 to standard power outlets.

**NOTICE**

- When connected correctly as described above, the Foot Control of either VarioSurg3 or Surgic Pro will operate the Link system.

## 4 Operating method

Turn ON the Power Switches of VarioSurg 3 and Surgic Pro.

### To Operate Surgic Pro (Fig. 8)

- 1) Hold down the SELECT key of VarioSurg3 for approximately 1 second.
- 2) "Surgic Pro" is displayed on the LCD Display of the VarioSurg 3.
- 3) Surgic Pro can be operated by the Foot Control.

Fig. 8

### To Operate VarioSurg3 (Fig. 9)

- 1) Hold down the SELECT key of VarioSurg3 for approximately 1 second.
- 2) The menu for VarioSurg3 is displayed in the LCD Display of VarioSurg3.
- 3) VarioSurg3 can be operated by the Foot Control.

Fig. 9

### CAUTION

- When using Surgic Pro and VarioSurg3 in the link mode, make sure that Power Switch on each unit is ON.
- When using Surgic Pro and VarioSurg3 in the link mode, do not turn OFF either of the Power Switches and do not pull out either of the AC power cords. Do not disconnect the SG Link Cable.
- To use only VarioSurg3, first confirm that neither VarioSurg3 nor Surgic Pro is operating, then disconnect the SG Link cable.

### NOTICE

- When "Surgic Pro" is displayed on the VarioSurg3 LCD Display, VarioSurg3 keys are disabled except the SELECT key.

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**NSK France SAS**
**NSK United Kingdom Ltd**
**NSK Asia Pte Ltd**