Pana-Max PLUS

Pana-Max PLUS

Please read this Operation Manual carefully before use, and file for future reference

MADE IN JAPAN
COM-T0344E

Intended Use

This handpiece is designed only for dental treatment use by qualified personnel.

CAUTION

- When operating this handpiece always consider the safety of the patient.
- Check for vibration, noise and overheating outside the patient's mouth before use. And if any abnormalities are found in use, stop using immediately and contact your dealer.
- Depressing the push-button while handpiece bur is in rotation will result in OVERHEATING of the handpiece head. Special caution must be exercised during use to keep cheek tissue AWAY from the push-button of the handpiece. Contact with cheek tissue may cause the push-button to depress and burn injury to the patient may occur.
- After long time use, handpiece may noisy. If this happened, change the cartridge. Do not use other than NSK products.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Use only NSK genuine cartridge. Turbine and cartridge should not be rebuilt or recycle by un-qualified NSK technician.

1. Specifications

Model	PAP-MU M4 PAP-MU B2	PAP-QD MU	PAP-SU M4 PAP-SU B2	PAP-QD SU
Head Type	Miniature		Standard	
Speed	380,000~450,000min ⁻¹		340,000~410,000min ⁻¹	
Spray Type	Four Port Water Spray			
Chuck	Push Button Chuck			
Bur Size	ISO1797-1 Type3			
	Short Shank Bur / Standard Bur		Standard Bur	
Drive Air Pressure	0.18MPa(1.8kgf/cm ²)	0.22MPa(2.2kgf/cm ²)	0.18MPa(1.8kgf/cm ²)	0.22MPa(2.2kgf/cm ²)
	-0.22MPa(2.2kgf/cm ²)	-0.25MPa(2.5kgf/cm ²)	-0.22MPa(2.2kgf/cm ²)	-0.25MPa(2.5kgf/cm ²)
Head size	Ø 10.6 mm X 12.4 mm		Ø 12.1mm X13.3 mm	

2. Connection / Disconnection to the Handpiece

PAP-MU M4/B2, PAP-SU M4/B2

- Connect
 - 1) Insert the handpiece into the hose connector.
 - 2) Tighten the Hose Nut. (Fig.1)

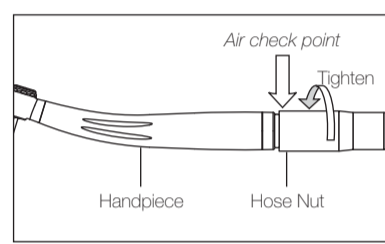
Fig. 1

- Disconnect
 - 1) Loosen the Hose Nut and pull out the hose.

PAP-QD MU, PAP-QD SU

- Connect
 - 1) Insert the coupling into the hose connector.
 - 2) Tighten the hose nut.
 - 3) Pull the connector ring
 - 4) Push into the handpiece into coupling.
- Disconnect
 - 1) Slide back the Connector Ring as shown in Fig. 2.

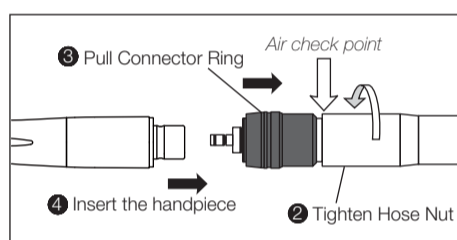
Fig. 2

CAUTION

During operation, do not pull the connector ring. (QD coupling)

Initial Setting - Service Engineer

Set the Air pressure at the arrowed point on Fig. 1 and Fig. 2. The setting values are on the specification table. Do not exceed the recommended air pressure.

Set the Multi Gauge between turbine and hose, and set and measure the air pressure. (Multi Gauge for Midwest 4 hole: Order No. Z109400, Multi Gauge for Borden 2 hole: Order No. Z109200)

3. Mounting and Removing the Bur

- Mounting
 - 1) Depress the push button and insert the bur into chuck.
 - 2) Release the button.
- Removing
 - 1) Depress the push button.
 - 2) Remove the bur.

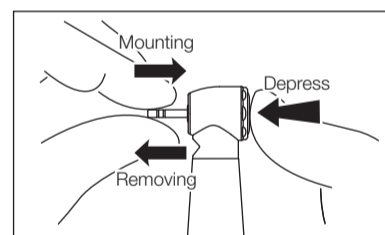
Fig. 3

CAUTION

- Use only ISO Standard burs. (Bur diameter Ø1.59 - Ø1.60 mm)
- Follow the recommended bur speed that a bur manufactures appoints. If the speed is lower than that, do not use.
- DO NOT use bent, worn, damaged or non-concentric burs. This may cause the cartridge's early breakage.
- Keep bur shank clean. Uncleaned shank may allow debris to build up inside chuck, causing weak bur retention.
- Insert the bur all the way into the chuck until it stops with the push-button fully depressed. Do not use short shank burs in Standard Head handpiece.
- Use only the bur specified as below.

Model	PAP-MU M4/B2, PAP-QD MU	PAP-SU M4/B2, PAP-QD SU
Mini. Attachment Length	9.0 mm	10.5 mm
Max. length	21 mm	25 mm
Max. processing dimension	Ø 2 mm	Ø 2 mm

4. Lubrication

Supply PANA SPRAY Plus, PANA SPRAY or Care3 Plus after each use and/or before autoclaving.

■ PANA SPRAY Plus/PANA SPRAY

PAP-MU M4/B2, PAP-SU M4/B2

- Mount the Arrow-Head Spray Nozzle into the spray port.
- Insert the Arrow-Head Spray Nozzle in the drive air tube of the handpiece and spray for approximately 2-3 seconds.

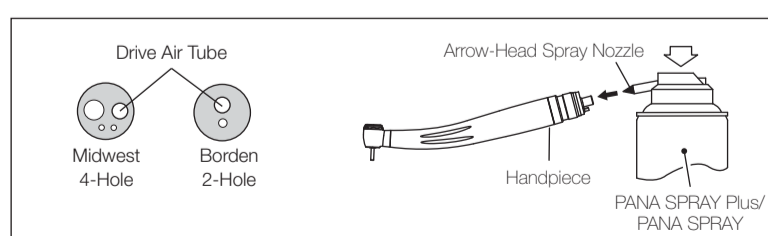
Fig. 4

PAP-QD MU, PAP-QD SU

- Mount the Spray Nozzle into the spray port.
- Insert the Spray Nozzle in rear of the handpiece and spray for approximately 2-3 seconds.

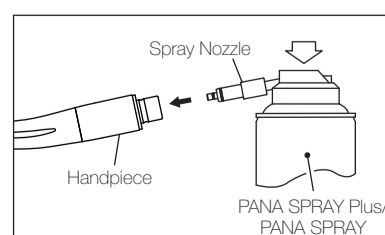
Fig. 5

CAUTION

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out of hand by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head.
- Hold the spray can upright.

■ Care3 Plus

NSK Care3 Plus automatic handpiece cleaning and lubrication system. After connecting the handpiece to the correct adaptor, activate the Care3 System per the Care3 System instructions.

Fig. 6

5. Sterilization

Autoclave sterilization is recommended.

Sterilization is required first time you use and after each patient as noted below.

■ Autoclave Procedure :

- 1) Scrub dirt and debris from the handpiece, and wipe clean with alcohol-immersed cotton swab or cloth. Do not use a wire brush.
- 2) Lubricate with PANA SPRAY Plus/PANA SPRAY/Care3 Plus. Refer to 4. Lubrication.
- 3) Insert into an autoclave pouch. Seal the pouch.
- 4) Autoclavable up to max. 135°C. ex.) Autoclave for 20 min. at 121°C, or 15 min. at 132°C.
- 5) Keep the handpiece in the autoclave pouch to keep it clean until you use it.

★ Sterilization at 121°C for more than 15 minutes is recommended by EN13060 or EN ISO17665-1.

CAUTION

- Do not autoclave the handpiece, even when it is bagged, together with other instruments, from which chemicals are not sufficiently cleaned. This is to avoid discoloration and/or damage to the plating of the handpiece.
- Do not wash, soak, or wipe off the handpiece with/in the oxidation potential solution (strong acid, super acid solution) or sterilized solution.

6. Changing Turbine Cartridge

- 1) Mount a dummy bur in the chuck.
- 2) Remove the Head Cap by Head Cap Wrench.
- 3) Push up the dummy bur to remove Cartridge.
- 4) Clean the head interior.
- 5) Align the Alignment Pin and insert Cartridge.
- 6) Mount the Head Cap. (Fig. 7)

★ Replacement Cartridge : SX-MU03 (for Miniature head) Order Code P1001250
SX-SU03 (for Standard head) Order Code P1002250

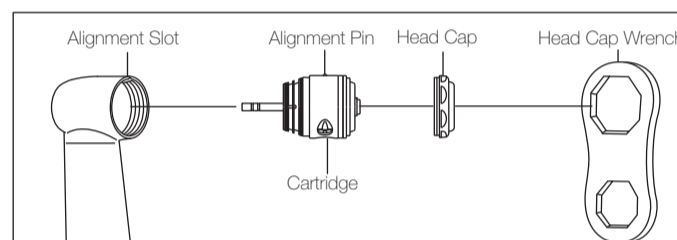
Fig. 7

CAUTION

- Make sure that two O-rings (Fig. 8) are in place on the Cartridge.

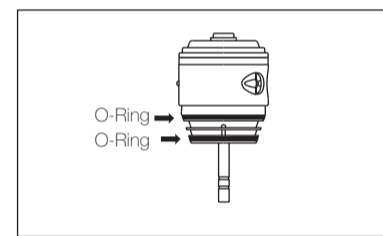
Fig. 8

7. Cleaning of Spray Ports

When Spray Ports are clogged, or spray does not exit evenly from each port, clean the ports as follows:

- 1) Remove the Cartridge.
- 2) Insert the Cleaning Wire straight into the Spray Port. (Fig. 9.)
- 3) Remove the debris.

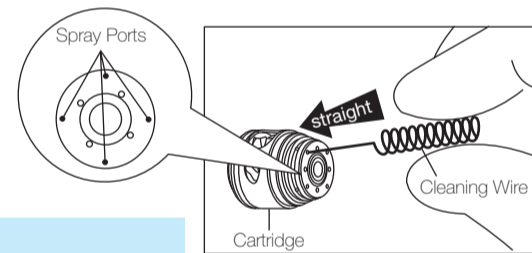
Fig. 9

CAUTION

Do not blow the air into Clean Head Hole.

8. Replacing the O-rings (QD Coupling)

Worn or damaged O-ring may cause air or water to leak. When water is found in the exhaust air or water leak from the coupled point, replace four O-rings as shown in Fig. 10.

★ Replacement O-ring Set (4pcs) : Order Code Y900592

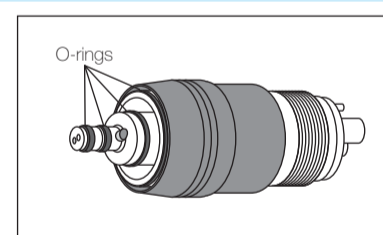
Fig. 10

9. Non-Retraction Valve (QD Coupling)

To prevent oral fluids drawn into the water line, PAP-QD MU, PAP-QD SU has Non-Retraction valve. If water is beginning to leaking from handpiece, replace non-retraction valve.

- 1) Remove the back-end gasket.
- 2) Replace Non-Retraction Valve.

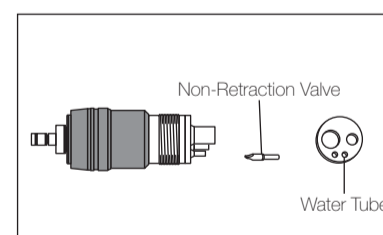
Fig. 11

CE The EU directive 93/42/EEC was applied in the design and production of this medical device.

NAKANISHI INC.
www.nsk-inc.com
700 Shimomiyata Kanuma-shi
Tochigi 322-8686,
Japan

NSK Europe GmbH
www.nsk-europe.de
Ely-Berthom-Strasse 8
65760 Eschborn,
Germany

NSK United Kingdom Ltd
www.nsk-uk.com
Office 5, Gateway 1000,
Arlington Business Park, Whittle Way,
Stevenage, SG1 2FP, UK

NSK America Corp
www.nskdental.com
1800 Global Parkway,
Hoffman Estates, IL 60192,
USA

NSK France SAS
www.nsk.fr
32 rue de Lisbonne
75008 Paris,
France

NSK Dental Spain SA
www.nsk-spain.es
C/ Modena, 43 El Soho-Európolis
28232 Las Rozas, Madrid,
Spain

NSK Oceania Pty Ltd
www.nskoceania.com.au
Unit 22, 198-222 Young St.
Waterloo, Sydney,
NSW 2017, Australia

NSK Middle East
www.nsk-inc.com
Room 6E-A-701, 7th Floor, East Wing No.6
Dubai Airport Free Zone,
PO Box 54316 Dubai, UAE

NSK Asia Pte Ltd
www.nsk-inc.com
1 Maritime Square,
#03-33 HarbourFront Centre,
Singapore 099253

Specifications are subject to change without notice.

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