OPERATION MANUAL

MADE IN JAPAN

Tip Power Guide

NAKANISHI INC. www.nsk-dental.com
700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

NSK Europe GmbH
Elly-Beinhorn-Strasse 8, 65760 Eschborn, Germany

Visit our
website

Specifications are subject to change without notice.

2021-10-25 CACB10XX 107

EN This operation manual describes the procedure of mounting/removing and reprocessing the tip. Carefully read the operation manual of the Ultrasonic System and the tips for other procedures.

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WARNING + B

EN • Always keep the proper file mounting length of ③. In case it is mounted with a different length, the file may be damaged due to variation of vibration nodes. • Be sure to use NSK files.

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CAUTION

EN • Only hold the tip at the base where it screws on to the handpiece. Otherwise there is a risk of injury to your hand or tip damage. • When attaching the tip, always wear sterilized gloves, and use a tip, handpiece, and tip replacement wrench that are all sterilized. • Make sure you don't twist the handpiece cord when tightening the tip.

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CAUTION

EN • Too tight mounting with the E-Tip wrench could cause breakage. Too loose mounting could reduce the power to the tip. • When mounting the tip to E11/E12/V30/V10, be careful not to excessively tighten the tip holder screw (*) as the holder screw may break. Tighten the screw gently by hand before cleaning and sterilization.

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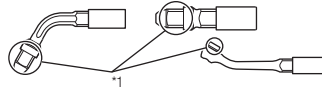
CAUTION + C

EN Appropriate vibration is not guaranteed unless the tip is inserted up to the dent mark (*2).

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CAUTION + D

EN • If tip is hit the healthy teeth or the prosthesis, tips could break. • The tip is very thin. Do not force tips but place them on teeth lightly.

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EN • Refer to the operation manual for the ultrasonic system that the handpiece and tip are to be connected to set the power level.

DE • Refer to the table 1 of this manual which defines the maximum power level of the tips according to the ultrasonic system which is being used.

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EN • Refer to the table 1 of this manual which defines the maximum power level of the tips according to the ultrasonic system which is being used.

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EN • During its operation, the whole tip vibrates due to ultrasonic waves. Do not bring any part of the tip into contact with soft tissue, gums or skin. Regardless of the use of coolant water the tip in vibration could cause a burn injury.

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EN • If the tip doesn't vibrate when the device is being activated, remove the handpiece from the patient's mouth, then restart the handpiece operation. If this does not improve the situation, the tip may have been loosened during use. Check that the tip is securely attached using the correct tool. • Never exceed the maximum power defined for the tip by this manual.

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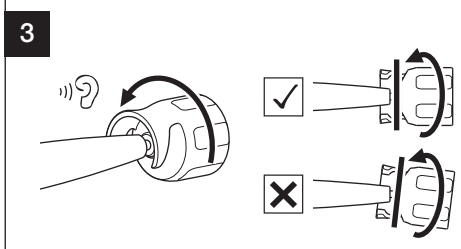
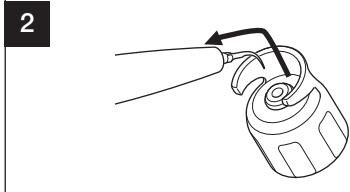
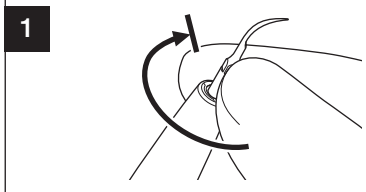
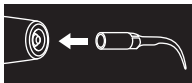
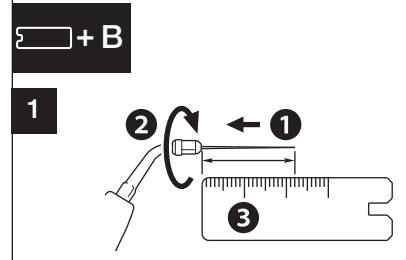
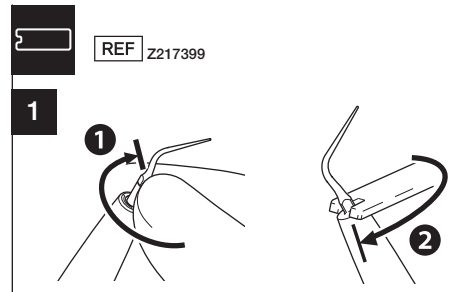
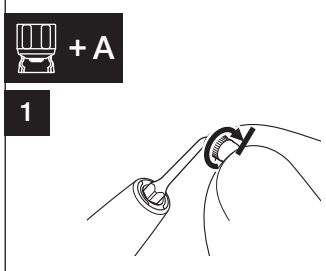
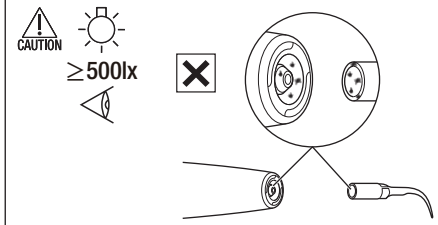
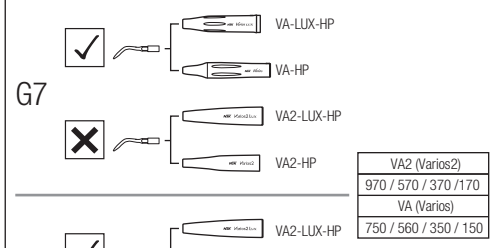
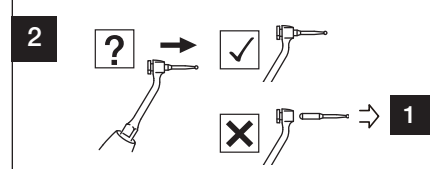
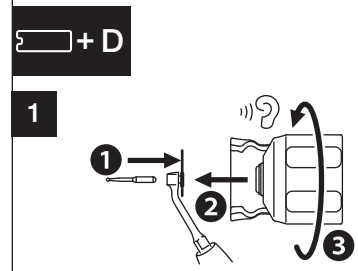
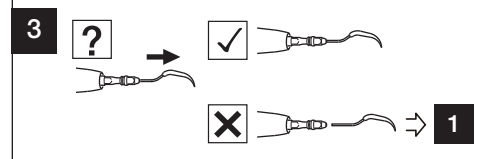
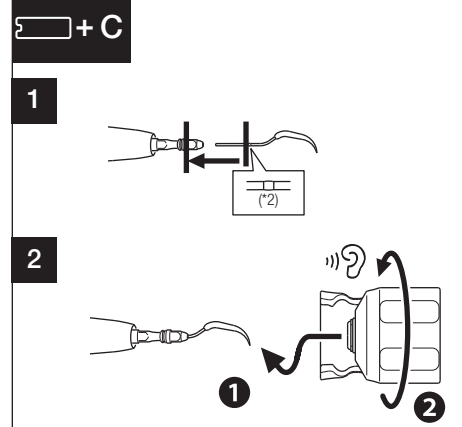
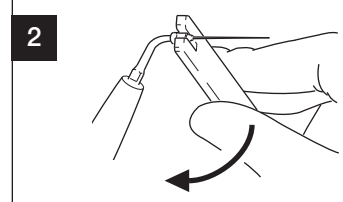
EN Immediately after a treatment, the tip should be cleaned, disinfected, sterilized and stored in accordance with this operation manual.

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EN Failure to properly maintain as below, the tip may cause infection, product failure, overheating leading to burn injuries:
- Be sure to clean the tip immediately (within 30 minutes) after use to remove residue.
- Be sure to disinfect the tip and the tip wrench, within 2 hours after cleaning.

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EN • Before using an alcoholic product for internal disinfection, the Tip must first be cleaned with a non-protein-fixing product (e.g. ALPRO WL-clean). • Use only cleaning/disinfectant products listed in this manual. • Follow any additional local directives, standards and guidelines for cleaning, disinfection, and sterilization.
• Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid medicine, solvent containing chlorine, benzene, or thinner. • Use clean and dry compressed air for cleaning. • Tips must be removed from the washer-disinfector immediately (within 1 hour) after the cleaning, disinfecting and drying cycle is complete to prevent corrosion. • After using washer-disinfector, dry the product completely. The remaining moisture causes internal corrosion etc. • Follow local rules, regulations and guidelines regarding the reprocessing of devices. • Do not autoclave the tip and the tip wrench with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments. • Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product. • Be sure to use sterilizers that can perform sterilization up to 135°C. In some sterilizers, the chamber temperature may exceed 135°C. Do not use these sterilizers as failure of the product could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures. • Clean the tip and the tip wrench prior to disinfection. If blood remains on the internal surface it can become clotted and cause product failure.
• Immediately after sterilization is complete (within 1 hour), remove the tip and the tip wrench from the sterilizer. Failure to do so may cause corrosion.

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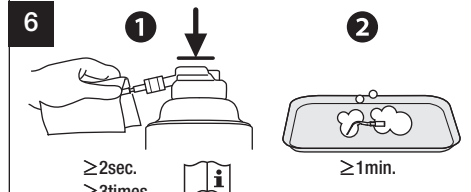
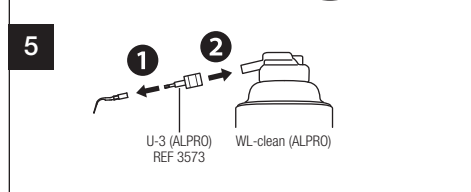
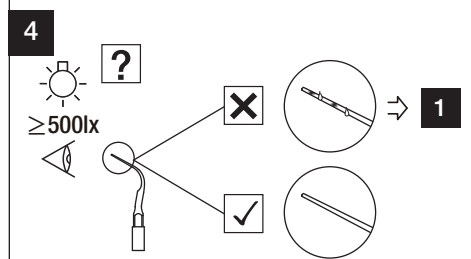
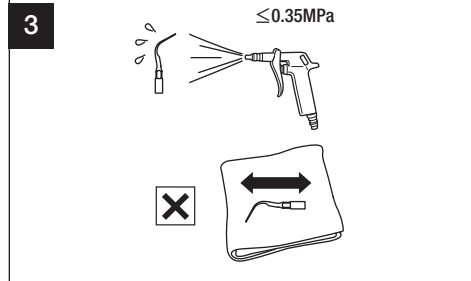
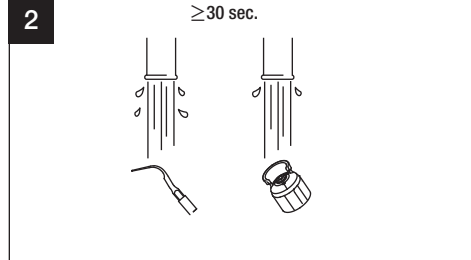
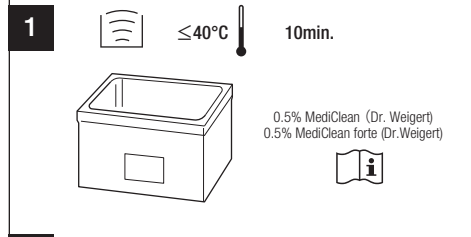
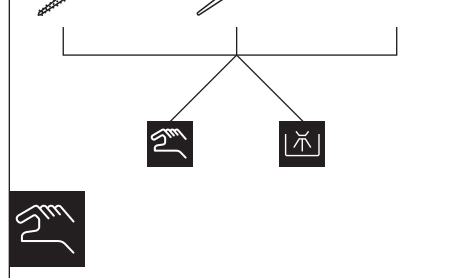
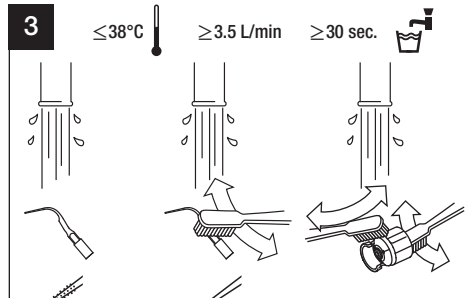
2

IC-100 (ALPRO) ≥5min.

EN Wait until cleaning/disinfectant solution works.

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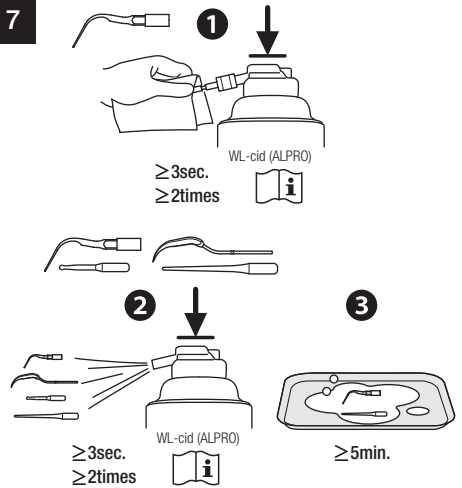
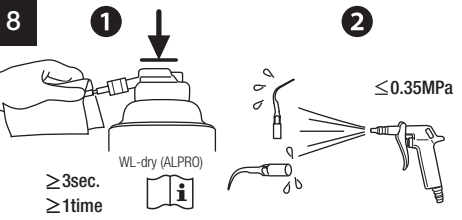
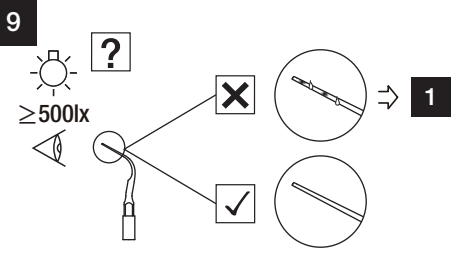
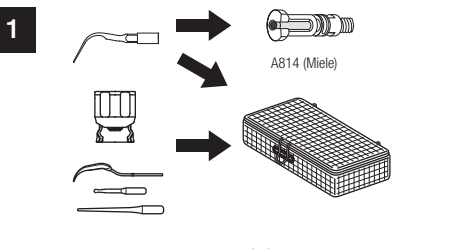
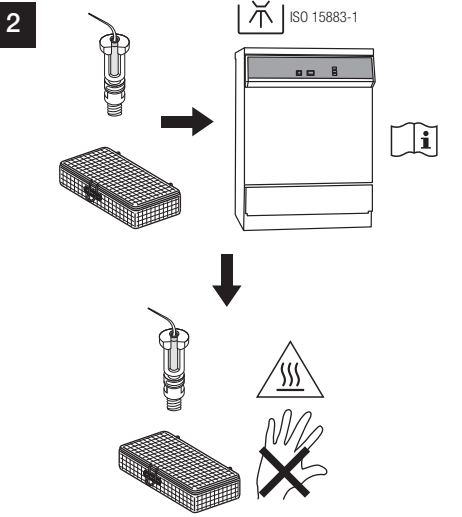
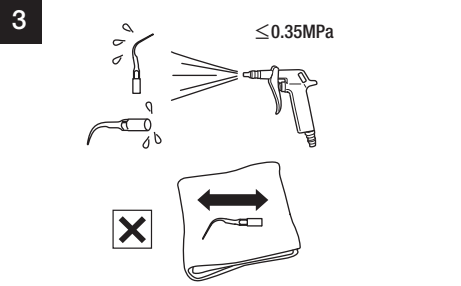
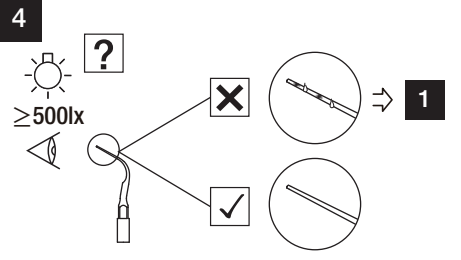
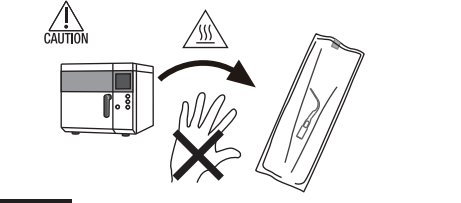
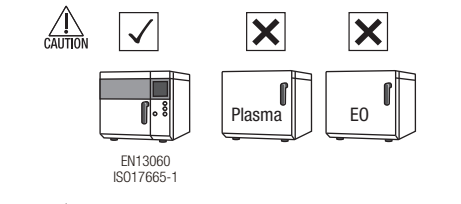
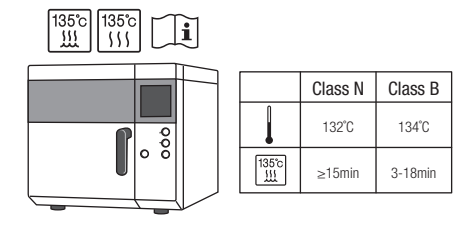
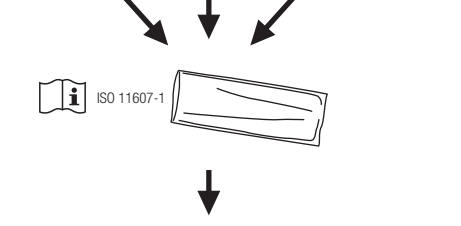
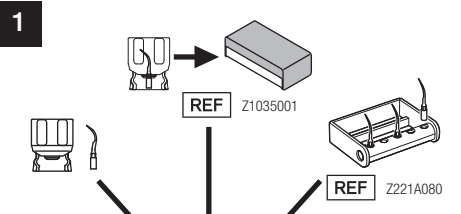
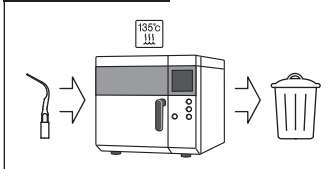
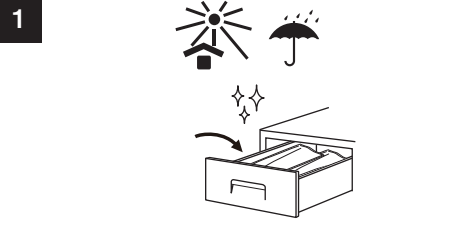
1

1

EN • In order to avoid any health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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EN * Avoid any risk of freezing of irrigation solution
When operated outside the above range, there is a risk of failure.

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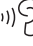
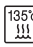
EN

- This tip is specially designed for the Ultrasonic System Varios970, Varios570, Varios370, Varios170, Varios750, Varios560, Varios350 and Varios150 by NSK. Do not use with other systems. • Be sure to always carefully read the operation manual for the Ultrasonic System before use to understand the usage and performance of the instrument.
- Be sure to always use with sufficient coolant water (except for dry tips, which do not require coolant water). Insufficient coolant water could cause a burn of the affected area or the overheating of the instrument.
- The "dry tip" utilizes heating to some extent for effective treatment. During or after its operation, the whole shaft tip could overheat. Do not bring any part of the tip into contact with your face, gums or skin. Ensure that the power setting of the Control Unit does not exceed the maximum power written on this manual. Adjust the power from the lowest level. During use of the tip, ensure that burn injury does not occur in or around the treatment area. Also ensure that the tip does not affect the treatment area and oral cavity. * E5, E6, E7, E8, G21, G22, G28 *
- Be sure to always use the tip at a setting not exceeding the maximum power level. If the tip is used in excess of the maximum power, the tip could break or excess cutting could occur. • Do not touch the top part of the tip to avoid an injury.

EN

- This product must only be used by a qualified personnel in accordance with the indications for use and the directions for use stated in the operation manual of the Varios Ultrasonic System it is attached to and this operation manual. Do not use it for any other purpose. • Operators and all others in the area must wear eye protection and a mask when operating this tip. • Prior to clinical use, inspect the tip outside the patient's oral cavity. Check for vibration, noise and overheating. If any abnormalities are found, stop using the tip immediately and contact your Authorized NSK Dealer.
- The tip is a consumable item and wears with use. If it is worn, abnormal noise, weak or no vibration, or damage may occur. Replace the tip with a new one when it is worn or loses diamond on the tip surface (only for diamond coated tips). Also, the tip can break due to metal fatigue. Replace with a new tip on a regular basis. • The tip may be discolored by repeated reprocessing. If the tip is discolored significantly, replace it with a new one. For the diamond coated tip, diamond on the tip surface may partially disappear and discoloration occurs. In this case, replace the tip with a new one. • Do not grind or bend the tip. It may break or may not vibrate during use. • Keep away from patients with cardiac pacemakers. • The tip is delivered in a non-sterile condition and must be autoclaved prior to use. • For irrigation, avoid use of ozonated water, oxidizing water (strongly acidic water, superacetic water, electrolyzed water, etc.), or hypochlorous solution. Doing so may lead to clogging, corrosion of metal parts, etc. • Perform suction while using the tip, since the tip may break depending on its use condition. • During vibration, do not hold the tip nor make contact with the tip for an extended time. • If there are abnormalities in water supply during operation, stop using the tip immediately and replace it with a new one. • If the tip cracks or breaks during use, immediately stop using the tip, and replace with a new tip. Operation with a damaged tip could injure the gums, periodontal tissue, or other locations. • Do not apply more pressure than required to the tip end. This could cause the tip to break or bend. • Be sure to activate the tip only after inserting the tip into the root canal. Allowing vibration in the air may cause the tip to break or come off due to loose tip retainer. (Root canal cleaning tip and U-File) • During endodontic use, to avoid either file or tips breakage, do not forcibly insert it into the root canal. When removing it from the canal, be sure to take it out gently. • If the water volume is too much, water may splash in the mouth or irritation becomes stronger. (P40/41) • Refer to the operation manual of the ultrasonic system for EMC information. • Report all serious events related to the product immediately to the manufacturer and respective national authority.

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